

Color-Coded Sizes


NEODRYS are color coded for quick identification. Available in two sizes: **Large** (blue) and **Small** (yellow). Packaged in convenient, easy-to-handle packs of 50.

Choice of Backing

Available in Reflective light-enhancing, mirror-like finish or Original white backing.

Small (Yellow)

- Original Backing
- Reflective Backing

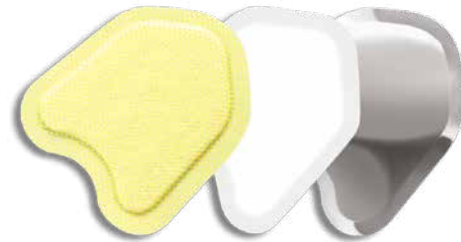

Large (Blue)


- Original Backing
- Reflective Backing




Method of Application

Select NEODRYS size which adequately covers the buccal mucosa. Insert as shown with color side against cheek and point to back of mouth. In a few seconds, NEODRYS will begin to adhere to the tissue and stay in place. **Important:** To remove without irritating tissue, release adhesion with ample water spray to the buccal side of the NEODRYS.


Applying NeoDrys

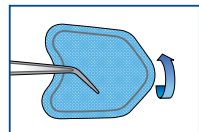

COLOR TO CHEEK

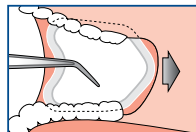

POINT TO BACK

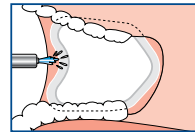

SPRAY TO RELEASE

NEODRYS®



Absorb, Protect & Reflect
Easier & More Effective Treatments.

- Sealants
- Composites
- Cementation
- Impressions
- Occlusion Marking
- Orthodontics
- CAD/CAM Prep
- Amalgam Preps
- Crown & Bridge
- Laser Dentistry
- Fluoride Applications
- Core Buildups

Also Available from Microcopy



FLAPS®

Comfortable, Kind Bite Tabs

- **Efficient:** Simple, self-sticking foam tabs attach quickly and are easy to position.
- **Reduces gagging.**
- **Comfortable:** Kids and adults love them.



BITE-CHEK®

Get a Handle on Articulation®

- **Easy-Grip handle**, no forceps.
- **No curling**, holds film flat.
- **Thin film** accurately targets high contact points.

www.NeoDrys.com



P.O. Box 2017
Kennesaw, GA 30156-9017
800-235-1863
770-425-5715

US7425664
EP1676552



NEODRYS®

Ultimate Parotid Saliva Control

Absorb, Protect & Reflect!



“NEODRYS are an amazing product, patients love how comfortable they are and I love how well they work!”

Alyssa K. DA

NEODRYS.COM





ABSORB, PROTECT & REFLECT.

Stop wasting money on pads and rolls that either fall apart or get saturated quickly. When you add up the reduction in botched procedures and wasted time — *NEODRYS cost less!*

ABSORB

- ◆ Super absorbent core—out performs messy cotton rolls and other absorbent pads.
- ◆ No messy leaks, even with the core fully saturated.



PROTECT

- ◆ Protects procedures from moisture contamination from the parotid gland for up to 15 minutes.
- ◆ Stays in place yet removes easily with water spray.
- ◆ Protective backing plus poly stiffener provide cheek retraction and protection.

REFLECT

- ◆ Choice of Original white or Reflective backing brightens the oral cavity, while retracting the cheek for an improved view.

NEODRYS FEATURES

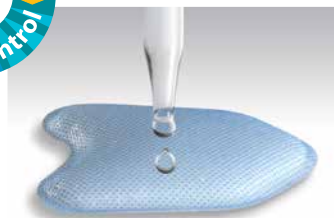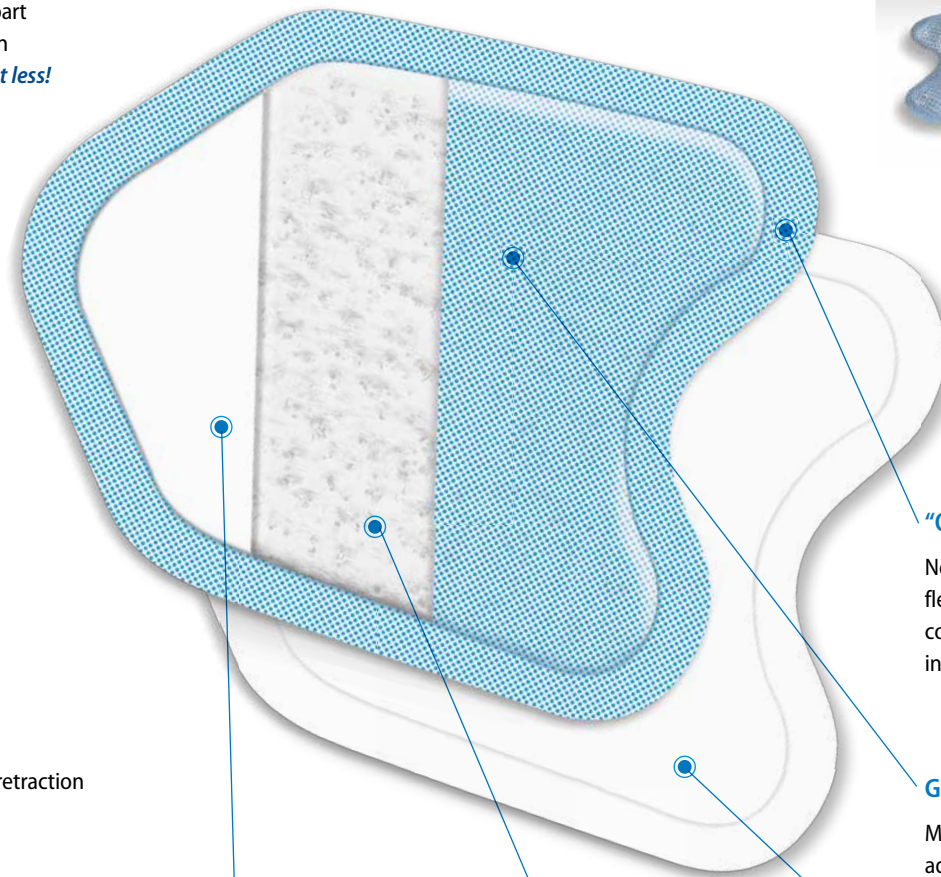

CORE STIFFENER

Poly stiffener insert provides optimum rigidity for cheek retraction.

SUPER ABSORBENT CORE

Acrylate polymer traps moisture as a gel when in contact with saliva.

"OUCHLESS" COMFORT EDGE®

New technology creates a soft, flexible edge with no sharp corners while retaining product integrity.

GENTLE TO THE BUCCAL MUCOSA

Medical-grade poly netting adheres gently to tissue.

PROTECTIVE BACKING

Choice of Original white or Reflective backing brightens the oral cavity while providing cheek protection.



Perry L. Parke
President/Owner

"No Questions Asked Guarantee."

"We unconditionally guarantee your total satisfaction with the quality and performance of all our products. If you are not completely satisfied, we will pick up the product and gladly refund your full purchase price."



ORIGINAL BACKING	50 PER PACK	TOTAL
SMALL	___ @ \$1325	\$ _____
LARGE	___ @ \$1325	\$ _____
REFLECTIVE BACKING	50 PER PACK	TOTAL
SMALL	___ @ \$1425	\$ _____
LARGE	___ @ \$1425	\$ _____
	US Shipping	\$ <u>6.95</u>
	(Add sales tax if delivered in GA)	\$ _____
	Total	\$ _____
	Canada Shipping	\$ <u>20.00</u>
	Canada Only - Free Shipping on Orders Over \$400	\$ _____
	Canadian Orders Apply GST and / or HST	\$ _____
	(Canadian Orders ship direct from Canada) Total	\$ _____

NEODRYS®

The same people who make NEODRYS, also bring you NEODIAMOND, NEOBURR & GAZELLE

Order Online:
NeoDrys.com

By phone:
800.235.1863

By fax:
770.423.4996



NEODIAMOND
America's #1 Selling
Diamond Bur



NEOBURR
Engineered for Safety



GAZELLE
Go the Distance

Microcopy
800.235.1863 | P.O. Box 2017 | Kennesaw, GA 30156-9017

facebook.com/microcopydental @microcopydental